



# Filtek™

Bulk Fill  
Flowable Restorative

## The bulk fill flowable that's easier to inject.

3M™ Filtek™ Bulk Fill Flowable Restorative is available in an ergonomic syringe designed for comfort and easier injecting.<sup>1</sup> This gives you more control and visibility for precise placement in deep restorations.

### Reduce bubbles and waste

Virtually no bubbles or material “run-on” during dispensing gives you better control.<sup>2,3</sup>

### No more guessing

The green syringe barrel indicates the remaining material volume. When your syringe is empty, only the white plunger shows.



### Bendable cannula

Easy-to-bend without kinking for better access.<sup>4</sup>

### Easy to hold and inject

Triangle finger plate and plunger make it quick and easy to your grip.<sup>5,6</sup>

## Key benefits

- Virtually no bubbles or material “run-on”<sup>2,3</sup>
- Easy-to-bend cannula for improved access<sup>4</sup>
- Ergonomic design is easy to hold and inject<sup>5,6</sup>
- Injectable, one-step placement up to 4 mm<sup>7</sup>
- Excellent adaptation and low shrinkage stress<sup>8</sup>

## Can be warmed for injection moulding

Warm up to 70°C for up to one hour, up to 25 times.<sup>9</sup>



## Posterior restoration made simple



Prepare the tooth and apply an adhesive e.g. 3M™ Scotchbond™ Universal Adhesive.



Dispense up to 4 mm of 3M™ Filtek™ Bulk Fill Flowable Restorative into the prepared cavity and cure, e.g. with 3M™ Elipar™ DeepCure-S LED Curing Light.



Place a 2 mm (minimum) capping layer of an aesthetic composite, such as a 3M™ Filtek™ Universal Restorative, and cure.

Clinical photos courtesy of Dr. Reich, Seefeld, Germany




3M Oral Care  
3M United Kingdom PLC  
Charnwood Campus  
10 Bakewell Road  
Loughborough  
Leicestershire, LE11 5RB  
+0800 626 578

3M Ireland Ltd  
The Lveagh Building  
Carrickmines Park  
Carrickmines  
Dublin 18  
Ireland  
+353 1 280 3555

3M.co.uk/Dental

## Indications

(Incremental up to 4 mm)

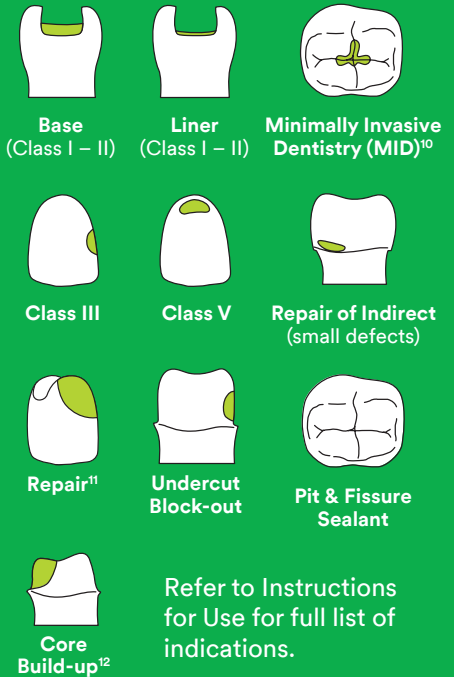

## Ordering information



### 3M™ Filtek™ Bulk Fill Flowable Restorative Refills

Syringe refills include: 2 – 2g syringes; 20 dispensing tips.

SHADE	ITEM NO.
A1	4862A1
A2	4862A2
A3	4862A3
Universal	4862U

ITEM NO.	PRODUCT INFO.
3740T	3M™ Filtek™ Bulk Fill Flowable Restorative Dispensing Tips, 19-gauge, Green (20)



### References

1. 3M Oral Care Internal Data. Easier to control delivery. Claim number 7636.
2. 3M Oral Care Internal Data. Virtually no bubbles. Claim Number 7642, 7647.
3. 3M Oral Care Internal Data. Virtually no run-on. Claim Number 7648.
4. 3M Oral Care Internal Data. Easy to bend the Tomodachi metal cannula. Claim Number 7656.
5. 3M Oral Care Internal Data. triangle shape plate allows for quick and easy orientation and grip. Claim Number 7654.
6. 3M Oral Care Internal Data. Easy to dispense material. Claim Number 7657.
7. 3M Oral Care Internal Data. up to 4mm placement. Claim Number 5138.
8. 3M Oral Care Internal Data. Low shrinkage stress. Claim Number 5328.
9. 3M Oral Care Internal Data. Warming. Claim Number 7665.
10. Including small, non stress-bearing occlusal restorations.
11. Repair of acrylic temporary and composite resin.
12. Where at least half the coronal tooth structure is remaining to provide structural support for the crown.

3M Elipar, Scotchbond and Filtek are trademarks of the 3M company. © 3M 2020. All rights reserved. 13432884-200210-G0111A